

DisCide[®] ULTRA Surface Disinfectant Instructions

If gross debris is present on the surface


CLEANING before disinfecting*



Step 1: Spray hard non-porous surface with DisCide Ultra Spray to help loosen bioburden.



Step 2: Wipe down the surface with a DisCide Ultra towelette to remove debris and bioburden.

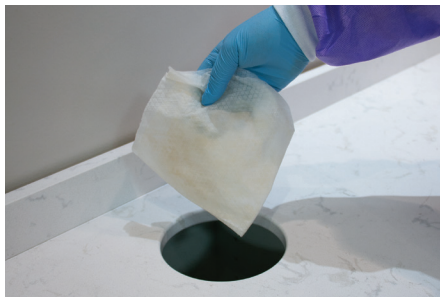

Step 3: Dispose of used DisCide Ultra towelette.

**If less than 5% bioburden present, 1-step cleaning and disinfecting can occur.*

If NO gross debris is present on the surface

DISINFECTING after cleaning*




Step 1: Use DisCide Ultra towelette or Spray DisCide Ultra Spray on the pre-cleaned surface.



Step 2: Leave the surface wet for one minute.



Step 3: Wipe down the surface with a DisCide Ultra towelette to remove any residue chemical left behind.

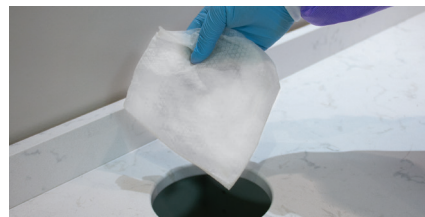

Step 4: Dispose of used DisCide Ultra towelette.



DisCide ULTRA Product Line

TOWELETTES



Ref# 10DIS
10 1/2" x 10" towelettes
60 count canister

Ref# 60DIS
6" x 6 3/4" towelettes
160 count canister

SPRAYS



Ref# 3565Q
Quart Sprayer

Ref# 3564
4 oz. Bottle

REFILLS



Ref# 3565G
Gallon Refill

ACCESSORIES



Ref# 3567
Empty quart bottle
with sprayer and
DisCide ULTRA label

Ref# 3567T
Quart spray trigger

Ref# 3519
Unlabeled empty
quart bottle with
sprayer



For more information, visit palmerohealth.com, email customerservice@palmerohealth.com or call 800.344.6424.